

FOCUS ON THE DOC

GRZEGORZ ZIĘTEK, DDS



Study finished in 1999 in Dental Institute of Jagiellonian University. Worked in Central Emergency Ambulatory of Jagiellonian University Clinic in years 2000 - 2005. Owner of private clinic and education center in Krakow since 2003. Since 2007 member of Polish Implantology Association (PSI), German Association of Oral Implantology (DGOI), International Congress of Oral Implantology (ICOI) and ICOI Europe. In 2013 member of the board of Polish Society of Implantology. Since year 2011 opinion leader of BEGO Implant Systems and their medical consultant in Poland. In year 2011 FELLOWSHIP of PSI, ICOI, DGOI. In year 2013 DIPLOMATE of PSI, ICOI, DGOI. Currently lecturing in Poland and abroad (lectured in Croatia, Japan, Vietnam, Albania). Author of publications in Poland and abroad.



For more information, contact :

Esacrom srl
VIA ZAMBRINI,6/A-40026 IMOLA(BO)
TEL. +390542643527 FAX
+390542482007
www.esacrom.com
comm@esacrom.com

Follow us on



@esacromsrl



Surgysonic®

20/10/20

ESACROM & ZIĘTEK present


«CrestaLift» Concept




By Dr. Grzegorz Ziętek, DDS (Pol)




STEP BY STEP PROCEDURE

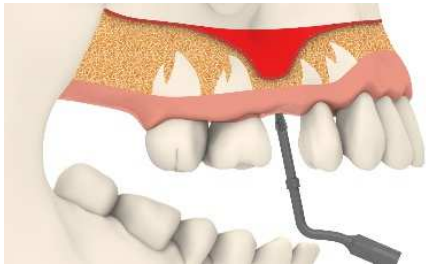


1- Positioning of the tip;

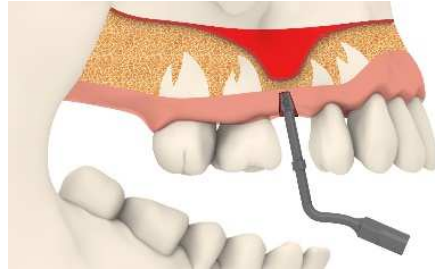


2- Hole realization;

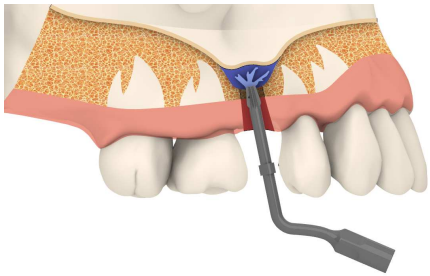


3- Tip lifting by using water at the top;

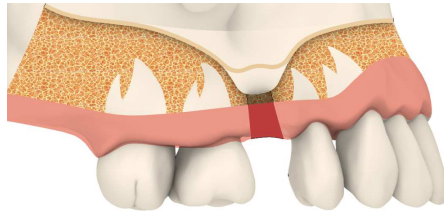

4- Sinus access diameter enlargement.

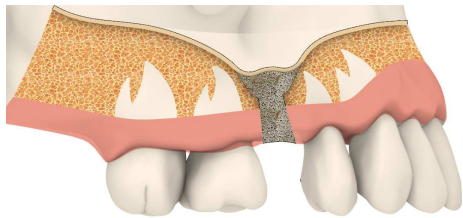

5- Preparation of the site and application of biomaterial;

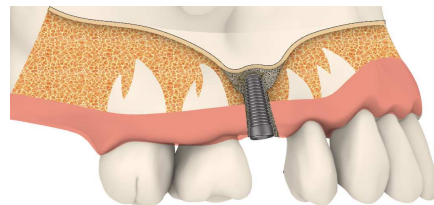

6- Positioning of the implant.

CLINICAL CASE



DEDICATED TIPS

ESOMCT

Micro-sharpened conical tip suitable for mini crestal elevation. Internal irrigation on the top.



U	40	} SWEEP MODE OR SURGERY MODE
V	80	
P	80	
MAX POWER	50	